

Snooooooze sleep positioning



Sleep Made
Simple:
Portable,
Adjustable,
& Built to Grow

LIVE LIFE

Support in 3 lying positions:

The Snooooooze system supports the user in supine (on their back), side and semi side-lying positions. This key feature enables us to accommodate and correctly position posture overnight, whilst maintaining a high level of comfort.

Supine lying:

To ensure correct positioning both during the day and night, we stabilize the pelvis. The pelvis is the foundation of good posture and is therefore the first area we evaluate when conducting an assessment. Using a component like the W Leg Trough, we aim to draw the legs into mid-line.

Side-lying:

The 'Body Roll' has heralded a new stage in side-lying, and can help achieve an excellent trunk and leg position for both left and right side-lying. It is most effective when the trunk has shortening sides and the legs are windswept. Alignment of the legs is achieved when a pillow is placed between the user's legs. The final key to achieving good posture is to support and position the head.

Sleep for everyone:

When the user struggles to sleep, it has a direct impact on the sleep patterns of every family member. Caregivers can be left exhausted, unable to think clearly and struggling to cope with their daily activities. By ensuring that the user has a good night's sleep, it also means that their family will enjoy the many benefits of sleep too.

Fitted Temperature Regulation Sheet

- Fitted sheet providing a comfortable and cool layer for sleep.
- Available in a range of mattress sizes.



	Cot	Single	King Single	Double
Code	7004-0031-000	7004-0032-000	7004-0033-000	7004-0034-000
Length	1450mm	2000mm	2100mm	2200mm
Width	800mm	1000mm	1080mm	1500mm

Receptor Sheet


- For secure positioning, this soft loop mattress sheet ensures that laterals and brackets stay in place, provided they have Velcro on their base. It is highly effective with both Curved Wedges and Tri-Fold Laterals.
- Sizes to suit a variety of mattresses.
- Machine washable.



	Cot	Single	King Single	Double
Code	7004-0021-000	7004-0022-000	7004-0023-000	7004-0024-000
Length	1300mm	1880mm	2030mm	1880mm
Width	690mm	920mm	1070mm	1380mm

Wedge

- Versatile support wedge for either the head or trunk.
- Where the neck tone is too great for the Star, a Wedge can be used up against the head, usually with the clients own pillow padding. Opposite positioning wedges may be needed on the trunk.
- Can also be used as a support against another component of the sleep system range.
- Removable and washable cover.

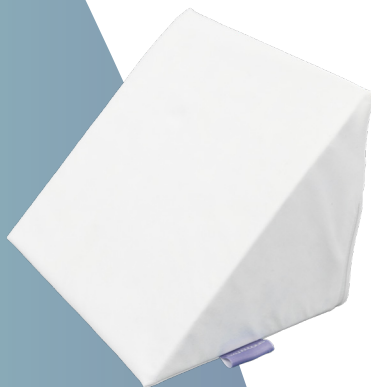

	Paediatric	Small	Large
Code	7004-0121-000	7004-0122-000	7004-0124-000
Length	100mm	200mm	405mm
Height	80mm	150mm	150mm
Width	80mm	150mm	50mm

Banana Cushion

- A multi-functional and versatile piece of sleep equipment. For example, it can be used as a short body roll, under a W Leg Trough, or under the neck.
- Removable and washable cover.
- A thermal regulation cover is available for this product.



	Paediatric	Large
Code	7004-0301-000	7004-0304-000
Diameter	100mm	180mm
Length	450mm	850mm
Width	120mm	200mm
Cover Code	7001-0301-000-C	7001-0304-000-C

W Leg Trough with Wrap



- W Leg Trough support cushion positions the legs, preventing rotation and reducing the risk of hip dislocation.
- Leg alignment is achieved with the whole leg supported and positioned correctly. The wrap (can be used with or without the wrap) prevents the client from crossing their legs and holds the position securely.
- W Leg Trough With Wrap – 3 Cells – The same great benefits of the W Leg Trough however this version has the added flexibility to adjust the filling within each area of the cushion independently, tailoring the support for the user. It provides opportunity to adjust for positions such as abduction, adduction, rotation and windsweeping.
- Removable and washable covers.
- A thermal regulation cover is available for this product.

	Paediatric	Small	Medium	Large	Tall
Code	7004-0401-000	7004-0402-000	7004-0403-000	7004-0404-000	7004-0405-000
3 Cell Code	7004-0411-000	7004-0412-000	7004-0413-000	7004-0414-000	7004-0415-000
Length	375mm	450mm	550mm	560mm	560mm
Width	410mm	490mm	580mm	800mm	1000mm
Height	10mm	180mm	240mm	240mm	240mm
Depth of W	80mm	150mm	200mm	200mm	200mm
Width within W	75mm	90mm	125mm	125mm	125mm
Width at top of W	75mm	90mm	100mm	100mm	100mm

Body Roll

- Long positioning piece offering full length support for users.
- Beneficial for side and semi side-lying positions, offering support either in front of or behind the user.
- Can also provide support between the legs for protection of the user's top hip joint.
- A thermal regulation cover is available for this product.



	Paediatric	Large
Code	7004-0331-000	7004-0334-000
Length	1300mm	2000mm
Diameter	90mm	150mm
Cover Code	7004-0331-000-C	7004-0334-000-C

Horseshoe

- Versatile support which can be used for multiple applications in supine and side-lying.
- A thermal regulation cover is available for this product.
- Removable and washable cover.



	One Size
Code	7004-0313-000
Length	1000mm
Diameter	210mm
Cover Code	7004-0313-000-C

Long Roll

- The Long Roll cushion can be used as a head, knee or back support.
- The Long Roll is a multi-functional cushion that can be used as a short body roll, under a W leg trough, under a neck or support other areas around the body.
- Removable and washable cover.
- A thermal regulation cover is available for this product.



	One Size
Code	7004-0322-000
Diameter	140mm
Length	210mm
Cover Code	7004-0313-000-C

T-Roll with Stabiliser

- Same great benefits of the standard T-roll, but with an added section to improve stability.
- Washable and removable cover.
- A thermal regulation cover is available for this product



	Paediatric	Large
Code	7004-0211-010	7004-0214-010
Diameter of Main Roll	150mm	250mm
Diameter of Centre Roll	150mm	250mm
Width Either Side of Roll	225mm	250mm
Cover Code	7004-0211-000-C	7004-0334-000-C

W Leg Trough with Wrap



- Adjustable head support with profiling points for added stability or alignment correction.
- For people with low to medium tone, the Star Head Support is ideal, as each of the points can be folded to create a shape which can be used to position the head as needed.
- Can be supplemented with wedges.
- Removable and washable covers.
- A thermal regulation cover is available for this product.

	Paediatric	Large
Code	7004-0801-000	7004-0804-000
Length Star Tip to Tip	560mm	690mm
Diameter Inner Circle	80mm	100mm
Cover Code	7004-0801-000-C	7004-0804-000-C

Tri-fold Positioning Bracket

- Can be used alone to directly support the body, or with other components from the Snooooooze range, such as the Body Roll.
- Has an internal bracket to ensure shape and support is maintained.
- Removable and washable cover.



	Paediatric	Small	Large
Code	7004-0111-000	7004-0112-000	7004-0114-000
Base Length	100mm	150mm	200mm
Height	85mm	130mm	200mm
Width	85mm	130mm	200mm

Twin Side Rolls

- The Twin Side Rolls and flap are a positioning element to centralise posture, often used by clients who have a feeding tube and require elevation during feeding.
- Can be used with other components of the sleep system range.
- Washable and removable cover.



	One Size
Code	7004-0820-000
Length at Roll(s)	900mm
Diameter of Roll(s)	150mm
Width of Top Between Rolls	460mm
Width at Bottom Between Rolls	360mm

Curved T-Roll with Wrap

- Same great benefits of the standard T-roll, but with a curved
- design for improved contouring around the underside of the legs.
- Supplied standard with wrap.
- Washable and removable cover.
- A thermal regulation cover is available for this product.



	One Size
Code	7004-0230-000
Diameter of Main Roll	250mm
Diameter of Centre Roll	250mm
Width Either Side of Roll	250mm

T-Roll

- T-Roll leg support and memory foam cushion supports the user's legs in abduction and inhibits adduction.
- For users who find the W Leg Trough unsuitable due to contracture at the knee,
- Washable and removable cover.
- A thermal regulation cover is available for this product.



	Paediatric	Small	Large
Code	7004-0201-000	7004-0202-000	7004-0204-000
Diameter of Main Roll	150mm	200mm	250mm
Diameter of Centre Roll	150mm	200mm	250mm
Width Either Side of Roll	225mm	250mm	250mm
Height of Centre Section	150mm	200mm	250mm



A large, stylized graphic of an eye, composed of two overlapping circles, serves as a background for the text. The circles are a darker shade of teal than the surrounding background.

LIVE LIFE

www.activerehab.net.au | support@activerehab.net.au | 03 5441 8966