

There are three alternatives for calf support: One Piece Calf Support (A), Separate Calf Supports (B), and Calf Strap (C).

A

B


C


One Piece Calf Support / Separate Calf Supports

One piece calf support is used when increased legrest stability is needed. It can also be used if a full supporting surface without gaps is required, for example if the user has involuntary leg movements.

Separate calf supports are used, for example, when there is a need of two different settings for the user's two legs.

One piece and Separate calf supports consist of calf pad (D) and calf pad bracket (E). These calf supports are continuously adjustable in height along the legrest tubes. The calf pads are adjustable 3 cm in depth, and the angle is continuously adjustable. The Separate calf supports can be laterally adjusted in a continuous range of 4,5 cm (F).

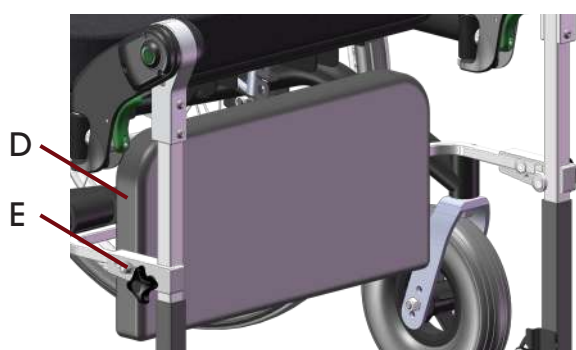

Calf Strap

Calf strap (G) can be used if a simpler calf support is desired. It is attached around the legrest tubes with velcro. Mounts (H) are used to maintain the height.


G

H



Measurements

Measurements in centimetres.

One piece calf support

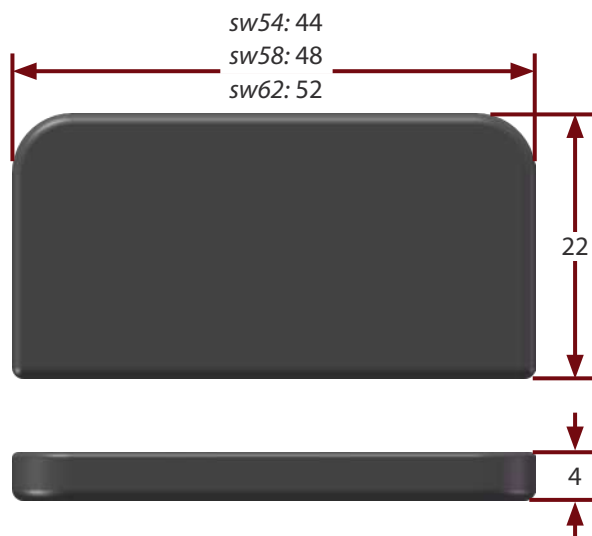

Separate calf supports

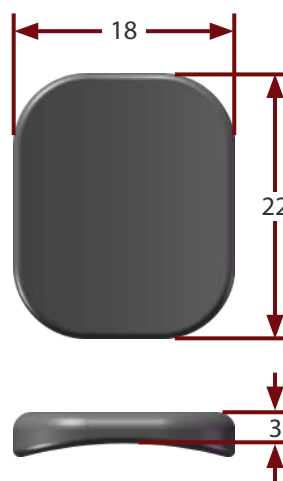

Article numbers

All article numbers are included in the Prescription Form, document number 97750-1.