

Headrest and Neck rest

There are two types of support for the patient's head / neck, Headrest (A) and Neck rest (B).



Headrest

The headrest gives a slightly larger support for the head and the shape provides a comfortable support to the sides. The lower part is slightly bevelled in order to not give pressure in the nape. The headrest can also be turned upside down if a thicker and rounder shape in the nape is desirable.

The headrest is available in three models:

Basic: With a headrest cushion without side adjustability (C).

Side: With a side adjustable headrest cushion (D).

Side+: With side adjustable headrest cushion and side support brackets (right and left) (E).




Neck rest

The neck rest (B) is slightly smaller and acts as a support in the nape of the more active user.

The neck rest is available in two models:

Side: With a side adjustable neck rest cushion (F).

Side+: With a side adjustable neck rest cushion and side support brackets (right and left) (G).

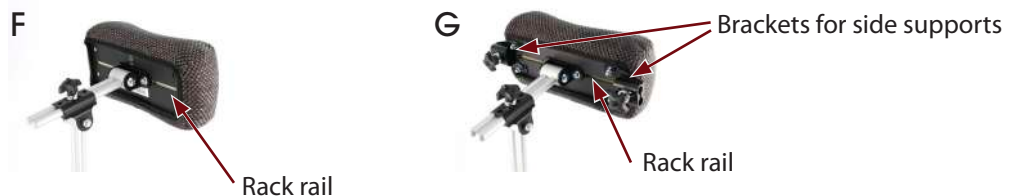

Side support

Side support is used when the patient has an increased need for lateral stability on one or both sides. The side supports are reversible and can be used on the left or right side. Side support cushions are the same for the headrest and neck rest. The bends, however, differ in the form.

The side support is available in two models:


Fix: Side support with fixed angle for headrest (H).

Vario: Side support with adjustable angle for headrest (I) and for neck support (J).



Settings

The headrest/neck rest consists of two parts; an extension bar (K) which is mounted in the bracket on the back of the wheelchair, and a headrest bar (L) with a cushion. The headrest/neck rest can be adjusted in height (24 cm) and depth (10 cm) using knobs. The angle of the cushion (M) and the angle between the headrest bar and extension bar (N) are adjustable without tools.

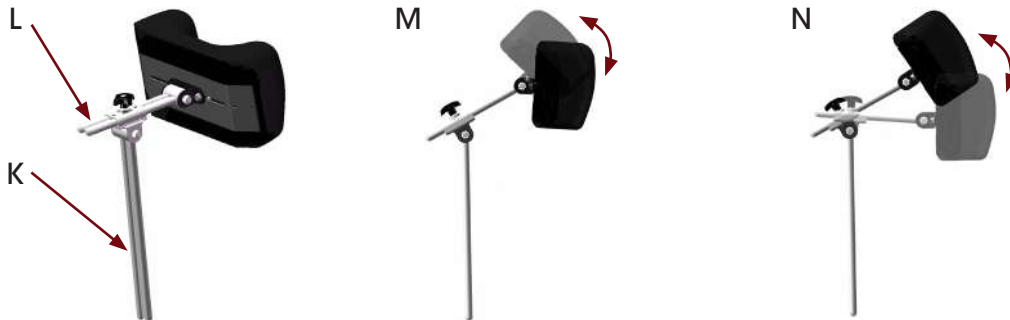

Profile stops (O) are available as accessories. These make it possible to always set the headrest/neck rest in the same position.




Measurements

Table 1 - Measurements - cushion [cm]

| Model | Height | Width | Thickness |
|--------------|--------|-------|------------------------|
| Headrest | 16 | 31 | 5 (centre of the back) |
| Neck rest | 13 | 28 | 6 (centre of the back) |
| Side support | 13 | 11,5 | 2,5 |

Keep in mind

It is possible to provide the headrest/neck rest with a longer headrest/neck rest bar (P) to increase the adjustability forward (10 cm).

 This is a special design which needs to be evaluated in terms of risk. Evaluate if the benefits of this special design overcome the additional risk.



Covers

Headrests and neck rests have grey Trevira CS fabric upholstery as standard. They can also be ordered in black disinfectable Dartex. All covers are removable and washable at 60° C. Covers can also be ordered separately.

Article Numbers

All article numbers are included in the Prescription Form, document number 97750-1.