



imedia

Immedia LeanOnMe

Relieving and supporting positioning wedges, triangles and cushions

Immedia LeanOnMe



LeanOnMe is a versatile collection of soft, mouldable cushions and with firm, stable wedges and triangles. Designed for flexibility, they can be used individually or combined to suit different needs. LeanOnMe offers personalised body support for both sitting and lying positions, while also helping to simplify caregiving tasks.

Fillings

Foam & beans

Filling for cushions



LeanOnMe cushions are filled with a balanced mix of granulated viscoelastic polyurethane foam residuals from mattress production and EPP beans (Expanded Polypropylene). This combination allows limbs to sink into the soft material and mould to individual needs. Both materials return to their original shape, providing resilient pressure distribution and load-bearing, even under heavy loads. The EPP beans are firm, quiet, water-resistant, and recyclable.

Key benefits:

- Soft, mouldable support for individual positioning
- Elasticity for pressure distribution
- Quiet materials for users with sensitive hearing
- Durable and long-lasting performance

Foam

Filling for wedges and triangles



LeanOnMe wedges and triangles use flexible, open-cell PU foam designed for comfort, support, and durability. Its excellent airflow helps keep users cool and comfortable, while its ability to distribute weight effectively helps reduce risk of pressure injury. This makes it ideal for both support and relaxation.

Key benefits:


- Firm, stable support for positioning
- Airflow for temperature regulation
- Effective weight distribution for pressure relief
- Durable material for long-term use

Covers

LeanOnMe covers come in three variants, Soft-Touch, Hygienic, and Non-Slip, designed to provide comfort, hygiene, and stability. All are STANDARD 100 by OEKO-TEX® certified, waterproof, breathable, and remain soft and bi-elastic even after repeated washing.

Soft-Touch

Available for cushions, wedges and triangles

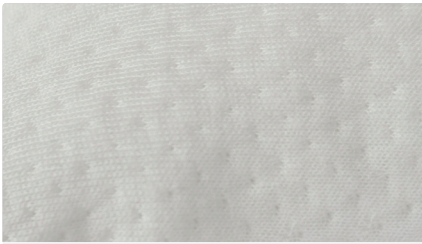

The Soft-Touch cover features a quilted, skin-friendly surface that feels pleasant and soft, allowing limbs to sink in comfortably. It also includes a non-slip polyurethane bottom to help keep wedges and triangles in place.

Key benefits:

- Air circulation
- Temperature regulation
- Barrier against dust mites, bacteria, fungi, and bed bugs
- Silent materials to minimise auditory stimuli

Hygiene

Available for cushions



The Hygienic cover has a soft, skin-friendly surface that is waterproof and easy to wipe clean. The material is pleasant and stretchable, allowing limbs to sink in comfortably.

Key benefits:

- Air circulation
- Temperature regulation
- Barrier against dust mites, bacteria, fungi, and bed bugs
- Silent materials to minimise auditory stimuli

Non-Slip

Available for wedges and triangles



The Non-Slip cover uses an anti-slip material with a skin-friendly surface to help keep products in place, regardless of the underlying surface.

Key benefits:

- Antibacterial properties

Material

LeanOnMe products are made using high-quality materials, including Lyocell (Tencel™) and Modal, combined with polyurethane membranes for durability and hygiene. Packaging uses recycled and recyclable materials.

LeanOnMe Cushions

Inner cushion: 95% polyester, 5% elastane

Filling: granulated residual polyurethane foam, EPP (Expanded Polypropylene) beans
Soft-Touch/Hygienic cover: 75% Lyocell (Tencel™ + Smartcel™ sensitive), 25% Modal (Tencel™). 100% polyester filling and polyurethane membrane

LeanOnMe Wedge and Triangle

Filling: PU comfort foam

Non-Slip material: 100% polyester with 100% polyurethane

Soft-Touch cover: 75% Lyocell (Tencel™ + Smartcel™ sensitive), 25% Modal (Tencel™). 100% polyester filling and polyurethane membrane

Washing instructions



Machine wash - max. temperature 80°C. Tumble drying - low temperature; max. 60°C. can be wiped off with detergent with pH-level between 5-9 or 70% disinfection solution or commercial and/or household cleaning agents. Remove inner cushion and foam before washing.

Immedia LeanOnMe Wedge

The wedges provide comfortable support for resting and personal hygiene. They offer firm, stable support and come with two covers: one with high friction to keep them in place, and another with a soft, skin-friendly surface made from sustainable materials, making LeanOnMe Wedges gentle for users with sensitive skin. Both covers feature non-slip material on the bottom to keep them in place on the mattress.

The wedges are ideal for individuals who need positioning support. Positioning with LeanOnMe Wedge helps protect vulnerable body areas, such as those at risk of pressure injuries. For users with mobility limitations, the wedge can enhance well-being and distribute pressure evenly, providing essential stability during positioning or hygiene routines.

LeanOnMe Wedge also helps create a safe, comforting environment, improving relaxation during rest and care routines.



Positioning options



LeanOnMe Wedges can be used in combination with two or more wedges to support where needed or as an aid for raising up from the bed to standing position.



LeanOnMe Wedges provide various ways to support the user in different resting positions. Besides supporting the back, the wedges can also be used to support the legs in a supine position.



LeanOnMe Wedges have two angles and can support a stable position when the patients back lies against the wedge or during personal hygiene tasks, or in combination with a LeanOnMe Triangle.

Product name	Size	Item no.
LeanOnMe Wedge XS + Soft-Touch Cover	W300xL265xH200 mm	IM53XSST
LeanOnMe Wedge XS + Non-Slip Cover	W300xL265xH200 mm	IM53XSNS
LeanOnMe Wedge S + Soft-Touch Cover	W300xL530xH200 mm	IM53SST
LeanOnMe Wedge S + Non-Slip Cover	W300xL530xH200 mm	M53SNS
LeanOnMe Wedge L + Soft-Touch Cover	W300xL800xH200 mm	IM5380LST
LeanOnMe Wedge L + Non-Slip Cover	W300xL800xH200 mm	IM5380LNS
Non-Slip Cover for IM5380LNS	W300xL800xH200 mm	IM530080LNS
Non-Slip Cover for IM53SNS	W300xL530xH200 mm	IM5300SNS
Non-Slip Cover for IM53XSNS	W300xL265xH200 mm	IM5300XSNS
Soft-Touch Cover for IM5380LST	W300xL800xH200 mm	IM530080LST
Soft-Touch Cover for IM53SST	W300xL530xH200 mm	IM5300SST
Soft-Touch Cover for IM53XSST	W300xL265xH200 mm	IM5300XSST

Immedia LeanOnMe Triangle

The triangles are versatile positioning aids designed to fill gaps between the body and mattress, providing targeted support that promotes relaxation and stability. They are ideal for addressing gaps under the stomach, hips, or beneath a pillow for extra head support. In this way, they help improve alignment and support, which can increase relaxation and well-being throughout the body.

The triangles help create a soothing, supportive environment that can promote restful sleep, distribute pressure on vulnerable areas, and foster stable support during rest and care activities. This makes LeanOnMe Triangles an essential aid for both users and caregivers.

The LeanOnMe Triangle XS offers a professional alternative to traditional supports and can fill various gaps. The LeanOnMe Triangle S is perfect for side-lying support, making personal care tasks easier and more comfortable. It can also provide stable support under the knees or feet.



Positioning options



The versatile LeanOnMe Triangle XS can be positioned in smaller gaps between the body and the mattress when a person is lying down, usually at the stomach, hips, or beneath the pillow.



Positioning the user well supported can contribute to relaxation and sleep.



LeanOnMe Triangles are ideal for providing a relaxing and stable side-lying position, making tasks such as care routines, dressing in bed, and wound care around the lumbar or pelvic areas easier for the carer.

Product name	Size	Item no.
LeanOnMe Triangle XS + Soft-Touch Cover	W200xL250xH100 mm	IM54XSST
LeanOnMe Triangle XS + Non-Slip Cover	W200xL250xH100 mm	IM54XSNS
LeanOnMe Triangle S + Soft-Touch Cover	W250xL400xH150 mm	IM54SST
LeanOnMe Triangle S + Non-Slip Cover	W250xL400xH150 mm	IM54SNS
Soft-Touch Cover for IM54XSST	W200xL250xH100 mm	IM5400XSST
Non-Slip Cover for IM54XSNS	W200xL250xH100 mm	IM5400XSNS
Soft-Touch Cover for IM54SST	W250xL400xH150 mm	IM5400SST
Non-Slip Cover for IM54SNS	W250xL400xH150 mm	IM5400SNS

Immedia LeanOnMe Roll

The roll can be bent, shaped and shaken so that it becomes fuller in some areas and thinner in others. This makes it adjustable to the user's specific needs.

When the user is lying in lateral side position, the LeanOnMe Roll can advantageously be shaped to support both the head, back, and legs. It can be well suited for users who benefits from tactile stimulation.



Positioning options



When used as a nest, the LeanOnMe Roll can reduce pain perception and risk of pressure injury on the shoulders, and will elevate the arms and hands. This can contribute to relieved breathing.



When hugged in the lateral position, the LeanOnMe Roll can provide well-being, and supports the user in a relaxing position.



When the LeanOnMe Roll is held into the stomach, it can contribute to a relieving and stabilising effect and support arms and shoulders.

Product name	Size	Item no.
LeanOnMe Roll S + Soft-Touch Cover	W330xL1000 mm	IM15112
LeanOnMe Roll L + Soft-Touch Cover	W330xL2300 mm	IM15113
LeanOnMe Roll S + Hygienic Cover	W330xL1000 mm	IM16112
LeanOnMe Roll L + Hygienic Cover	W330xL2300 mm	IM16113
Soft-Touch Cover for IM15112	W330xL1000 mm	IM10512
Soft-Touch Cover for IM15113	W330xL2300 mm	IM10513
Hygienic Cover for IM16112	W330xL1000 mm	IM10612
Hygienic Cover for IM16113	W330xL2300 mm	IM10613

Immedia LeanOnMe Channel

The channel-sewn cushion is flexible and offers support and relief. The cushion can be rolled or shaped as needed.

The LeanOnMe Channel cushion can be positioned under the leg(s), between the legs, under the calves to evenly distribute pressure on the heels, or under an arm to ease shoulder blades and/or elbows. When in a lateral position, the cushion can be placed under the user's back or between their legs to help distribute pressure and reduce strain on exposed areas. While sitting, the cushion can be placed in the back as support.



Positioning options



When a LeanOnMe Channel is positioned under both legs, it can reduce oedema by elevating the legs. It can also alleviate pain perception and lower the risk of pressure injury.



The LeanOnMe Channel can be used for elevating legs or for pressure distribution on the ankle and/or heel.



In a lateral position a LeanOnMe Channel cushion can be placed between the legs to help exposed areas, such as inner knees and ankles, from pressure.

Product name	Size	Item no.
LeanOnMe Channel + Soft-Touch Cover	W750xL750 mm	IM15108
LeanOnMe Channel + Hygienic Cover	W750xL750 mm	IM16108
Soft-Touch Cover for IM15108	W750xL750 mm	IM10508
Hygienic Cover for IM16108	W750xL750 mm	IM10608

Immedia LeanOnMe Wing

The backrest cushion with a wing is used in a lateral position. The user lies on the wing so that their weight keeps the cushion in place. Then, the user's back is comfortably supported in a stable position.

The LeanOnMe Wing cushion can also be used for supine positioning; the pillow can be positioned to help distribute pressure on exposed parts of the body.



Positioning options



When positioned at the back in a lateral position with the wing beneath the user, the cushion remains secure and prevents the user from tilting backwards.



The LeanOnMe Wing cushion can protect exposed body parts from pressure from for example bed rails.



Placed underneath the arm in a supine position, the LeanOnMe Wing helps reduce pressure and provides a pain-relieving effect.

Product name	Size	Item no.
LeanOnMe Wing + Soft-Touch Cover	W450xL800 mm + wing 200 mm	IM15107
LeanOnMe Wing + Hygienic Cover	W450xL800 mm + wing 200 mm	IM16107
Soft-Touch Cover for IM15107	W450xL800 mm + wing 200 mm	IM10507
Hygienic Cover for IM16107	W450xL800 mm + wing 200 mm	IM10607

Immedia LeanOnMe Mini

A small, versatile support cushion placed where smaller gaps occur between the body and mattress when a person is positioned; for example, under the lower back, at the ankle, and the knee.

Using a LeanOnMe Mini can have a relaxing effect on the body part. It can also be used to open a hand affected by spasticity.



Positioning options



A LeanOnMe Mini placed under the lower back to evenly distribute pressure on the pelvis.



Use a LeanOnMe Mini for relieving oedema by elevation and open the hand.



The LeanOnMe Mini can be placed where smaller gaps occur.

Product name	Size	Item no.
LeanOnMe Mini + Soft-Touch Cover	W150xL300 mm	IM15100
LeanOnMe Mini + Hygienic Cover	W150xL300 mm	IM16100
Soft-Touch Cover for IM15100	W150xL300 mm	IM10500
Hygienic Cover for IM16100	W150xL300 mm	IM10600

Immedia LeanOnMe Basic

Traditional positioning cushions in six shapes and sizes, The cushions can be used in positioning to support the back, arms, legs, knees, and ankles.

The LeanOnMe Basic positioning cushions can also be used to reduce pressure on exposed body parts by distributing pressure evenly.



Positioning options



LeanOnMe Basic cushions can be used in the supine position to support the body or extremities.



A LeanOnMe Basic cushion can be held on the stomach to relieve pain perception and/or improve well-being.



Alignment and support of the arm and hand to minimise the risk of contractures, oedema, and pressure.

Product name	Size	Item no.
LeanOnMe Basic XS +Soft-Touch Cover	W300xL400 mm	IM15101
LeanOnMe Basic S + Soft-Touch Cover	W400xL600 mm	IM15102
LeanOnMe Basic M + Soft-Touch Cover	W500xL700 mm	IM15103
LeanOnMe Basic L + Soft-Touch Cover	W250xL800 mm	IM15104
LeanOnMe Basic LW + Soft-Touch Cover	W450xL800 mm	IM15105
LeanOnMe Basic XL + Soft-Touch Cover	W550xL900 mm	IM15106
LeanOnMe Basic XS + Hygienic Cover	W300xL400 mm	IM16101
LeanOnMe Basic S + Hygienic Cover	W400xL600 mm	IM16102
LeanOnMe Basic M + Hygienic Cover	W500xL700 mm	IM16103
LeanOnMe Basic L + Hygienic Cover	W250xL800 mm	IM16104
LeanOnMe Basic LW + Hygienic Cover	W450xL800 mm	IM16105
LeanOnMe Basic XL + Hygienic Cover	W550xL900 mm	IM16106
Soft-Touch Cover for IM15101	W300xL400 mm	IM10501
Soft-Touch Cover for IM15102	W400xL600 mm	IM10502
Soft-Touch Cover for IM15103	W500xL700 mm	IM10503
Soft-Touch Cover for IM15104	W250xL800 mm	IM10504
Soft-Touch Cover for IM15105	W450xL800 mm	IM10505
Soft-Touch Cover for IM15106	W550xL900 mm	IM10506
Hygienic Cover for IM16101	W300xL400 mm	IM10601
Hygienic Cover for IM16102	W400xL600 mm	IM10602
Hygienic Cover for IM16103	W500xL700 mm	IM10603
Hygienic Cover for IM16104	W250xL800 mm	IM10604
Hygienic Cover for IM16105	W450xL800 mm	IM10605
Hygienic Cover for IM16106	W550xL900 mm	IM10606

Immedia LeanOnMe Ring

The ring cushion offers stable, relieving support when a standard pillow is unsuitable.

It is ideal for users at risk of occipital pressure ulcers, as it distributes pressure evenly. It can also be used in a half-seated bed or armchair to provide a suitable headrest for users with poor head control.

The LeanOnMe Ring can easily be used on trips to relieve the pain perception of sore neck muscles and vertebrae.



Positioning options



For distributing pressure on ears, and/or pain relief of neck muscles when lying in lateral position.



To reduce the risk of pressure injury on the back of the head and improve head control in a lying position.



Use the LeanOnMe Ring to support the head in a sitting position.

Product name	Size	Item no.
LeanOnMe Ring S + Soft-Touch Cover	Ø400 mm	IM15110
LeanOnMe Ring M + Soft-Touch Cover	Ø500 mm	IM15111
LeanOnMe Ring S+ Hygienic Cover	Ø400 mm	IM16110
LeanOnMe Ring M + Hygienic Cover	Ø500 mm	IM16111
Soft-Touch Cover for IM15110	Ø400 mm	IM10510
Soft-Touch Cover for IM15111	Ø500 mm	IM10511
Hygienic Cover for IM16110	Ø400 mm	IM10610
Hygienic Cover for IM16111	Ø500 mm	IM10611



Etac is a world-leading developer of innovative assistive devices and patient handling equipment. Since 1973, we have been committed to improving quality of life for the individual, family members and caregivers.

For the intended purpose of the products and latest information, please visit www.etac.com

Etac

+46 371 587 330

info@etac.se

www.etac.com

etac[®]

Creating Possibilities